

Jupiter Portable Magnifier™

User Manual



Copyright © 2019, 2026

American Printing House for the Blind.

All rights reserved. Printed in the United States of America.

This publication is protected by Copyright and permission must be obtained from the publisher prior to any reproduction, storage in a retrieval system, or transmission in any form or by any means, electronic, mechanical, photocopying, recording, or otherwise.

For information regarding permissions, write to:

American Printing House for the Blind
Resource Services
1839 Frankfort Avenue
Louisville, KY 40206

All efforts have been made to ensure that information contained in this document is correct at the time of issue. Neither APH nor its representatives assume any responsibility for errors or omissions, nor is any liability assumed for damages resulting from the use of the information contained herein.

The Jupiter was developed by Vispero and is distributed by APH. Jupiter Portable Magnifier is a trademark of APH.

Contents

A. Precautions	4
B. Package	6
C. Instructions Before Use	7
D. Opening the Jupiter.....	8
E. Closing the Jupiter.....	9
F. Camera Position	10
G. Configuration	11
H. Hot Keys & Features	19
I. Plug&Play Smart Premium	24
J. Technical Specifications.....	26

A. Precautions

Please read this manual carefully to ensure safe and proper use of the product.

! Important !


- Negligent use or misuse of the product may cause a fire, personal injury, or serious damage.
To prevent electric shock or fire:
 - Do not use damaged power plug or unsuitable outlet.
 - Do not touch power plug with wet hands.
- Misuse of the product may result in personal injury or damage to the unit.
- Use only the adapter supplied with the product. Other adapters may damage the unit.
- Do not use this unit in moist environments, such as a bathroom or shower room.
- Do not place heavy objects on the unit.
- Do not attempt to disassemble, remodel, or repair the unit. Doing so may damage the unit and will void the warranty.

A. Precautions


- After six (6) months usage, the lifespan of the battery may be shortened.
- Keep the product out of direct sunlight and extreme temperatures, as the display may not work properly.
 - The recommended operating temperature is between 0 ~ 35 °C (32 ~ 95 °F)

B. Package


Thank you for choosing Jupiter Portable Magnifier™!
Please ensure the following items are in the Jupiter package.

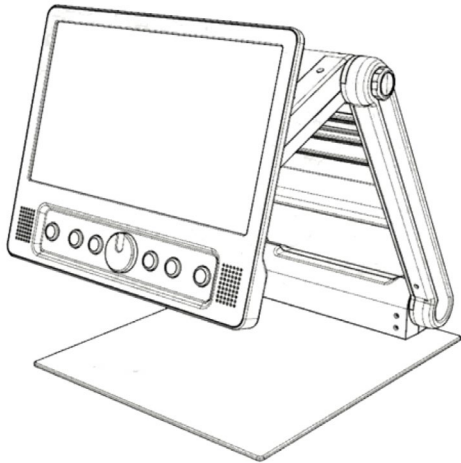


1. Jupiter Portable Magnifier™

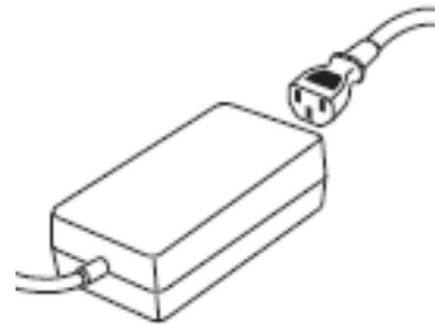

2. Power adapter

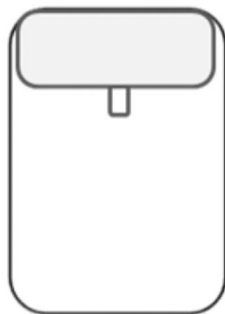

3. Carrying pouch

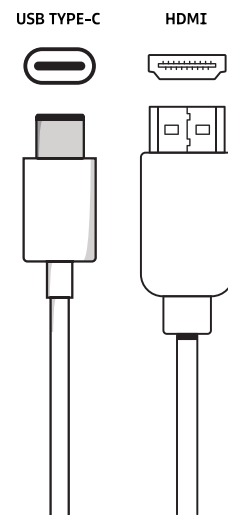

4. USB-C to HDMI cable
for Plug&Play feature

C. Instructions Before Use

- The battery must be fully charged prior to the initial use.
Please connect the supplied power adapter to the DC input jack to charge the battery.
- Remove the LCD protection film before using the unit.
- Be careful not to allow foreign material to adhere to the camera lens.

D. Opening the Jupiter



1. Place the folded Jupiter on a stable surface such as a table.
2. Press on the base plate with one hand, while grasping the top of the body frame with the other.
3. Lift the arm until it locks in place.
4. Put both hands on the left and right sides of the LCD monitor.
5. Pull the LCD monitor towards you.
6. Push the LCD monitor up using both hands and adjust it to a suitable position.

E. Closing the Jupiter



1. Put both hands on the left and right sides of the LCD monitor.
2. To close the Jupiter, push down the LCD monitor using both hands.
3. Push the LCD monitor toward the main body frame.
4. Place both hands on top of the body frame.
5. Lower the top of the body frame until it is closed.


F. Camera Position



You can switch between modes by rotating/adjusting the camera pointer (camera positioning wheel).

1. Reading mode (downward)
2. Distance-view mode (front-facing)
3. Self-view mode (mirror mode)


Hardware Description

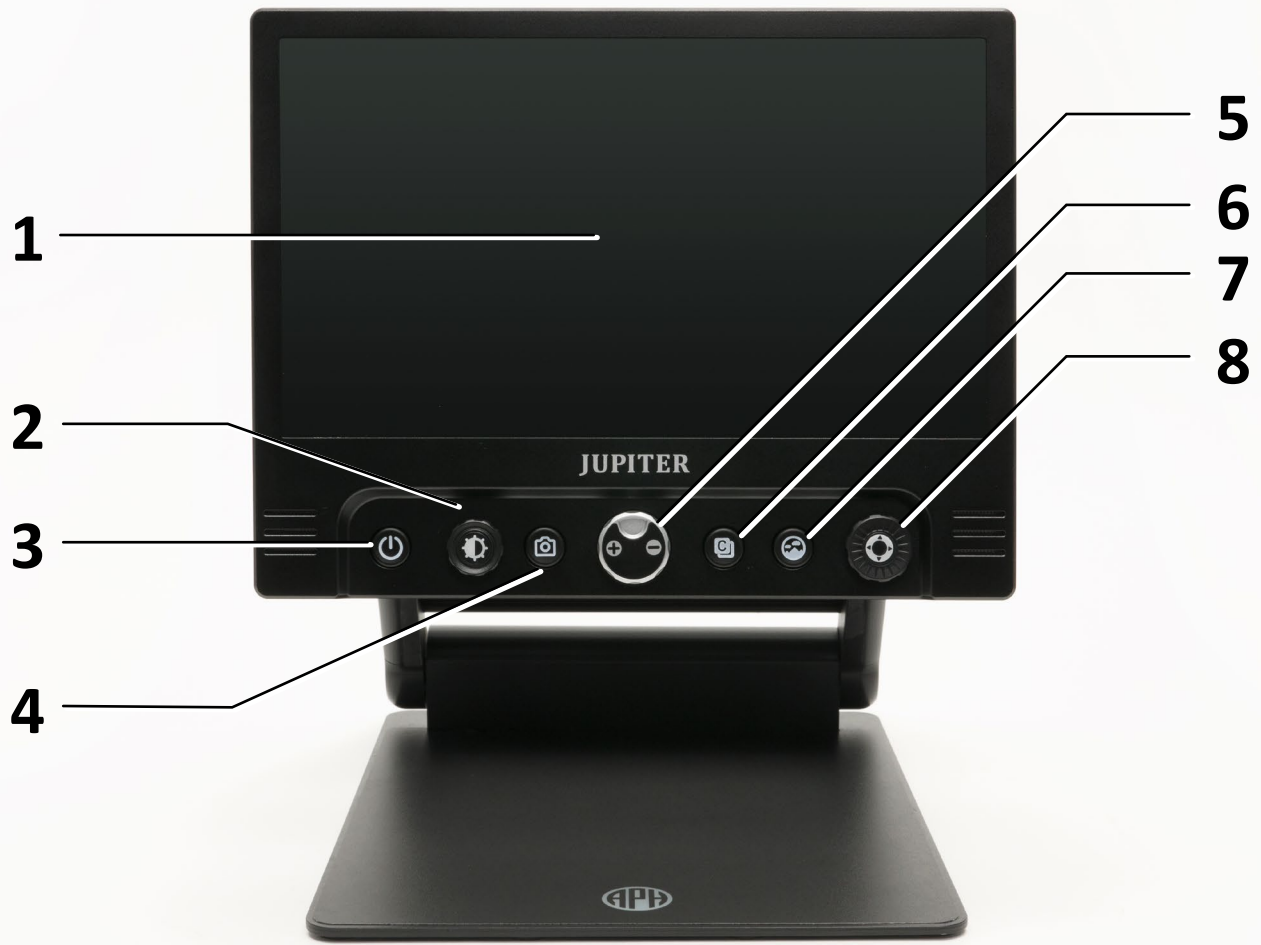

- | | |
|--|-----------------------------------|
| 1. LCD screen | 5. Zoom wheel |
| 2. Image optimizer wheel
(adjusting brightness) | 6. Color mode button |
| 3. Power on/off button | 7. Natural color button |
| 4. Snapshot button | 8. Super function key (A Jog Key) |

G. Configuration, Cont'd.

Hardware Description

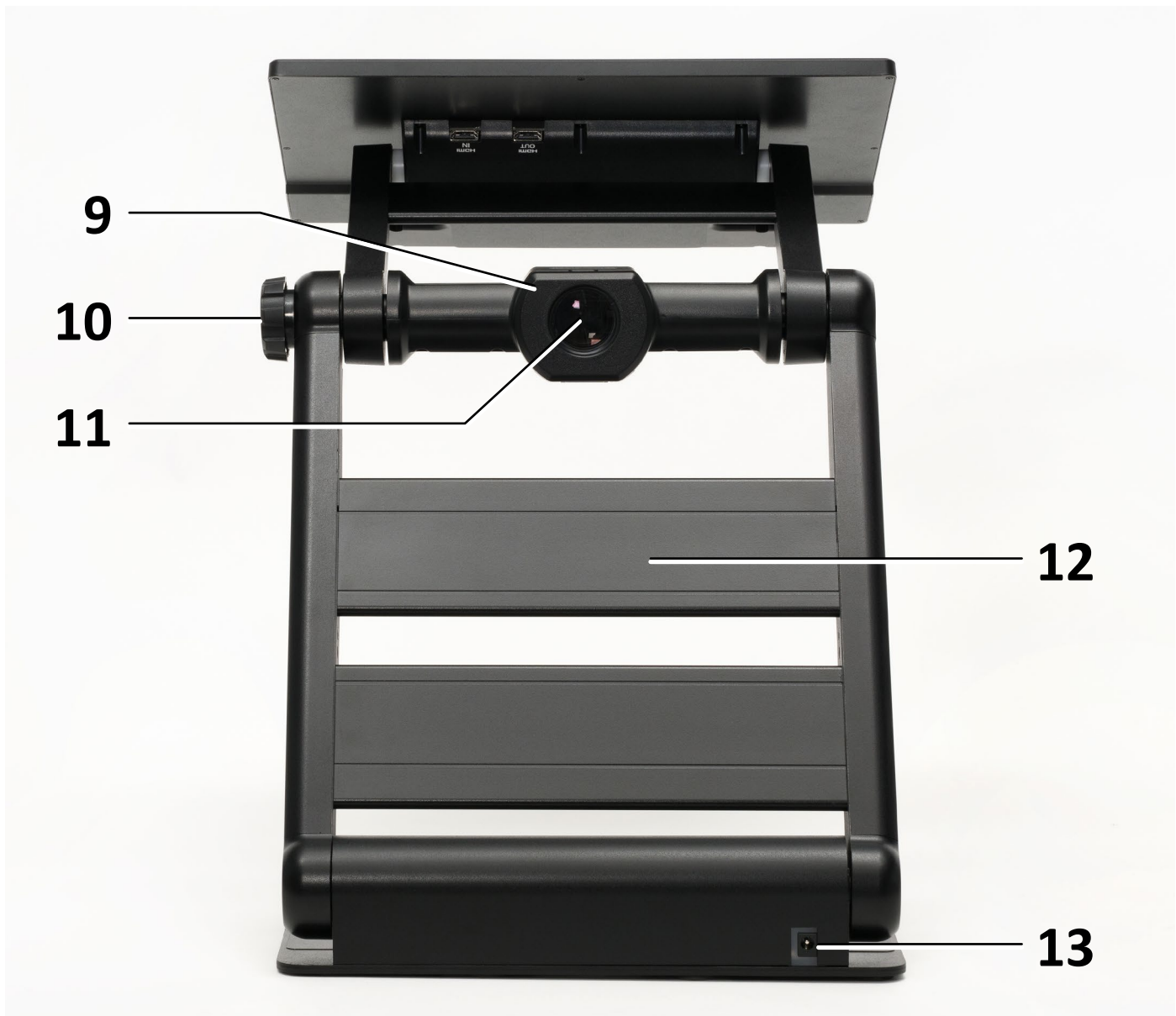

9. Camera


10. Camera location adjuster wheel

11. Close up lens

12. LED lights

13. DC jack

External Connections

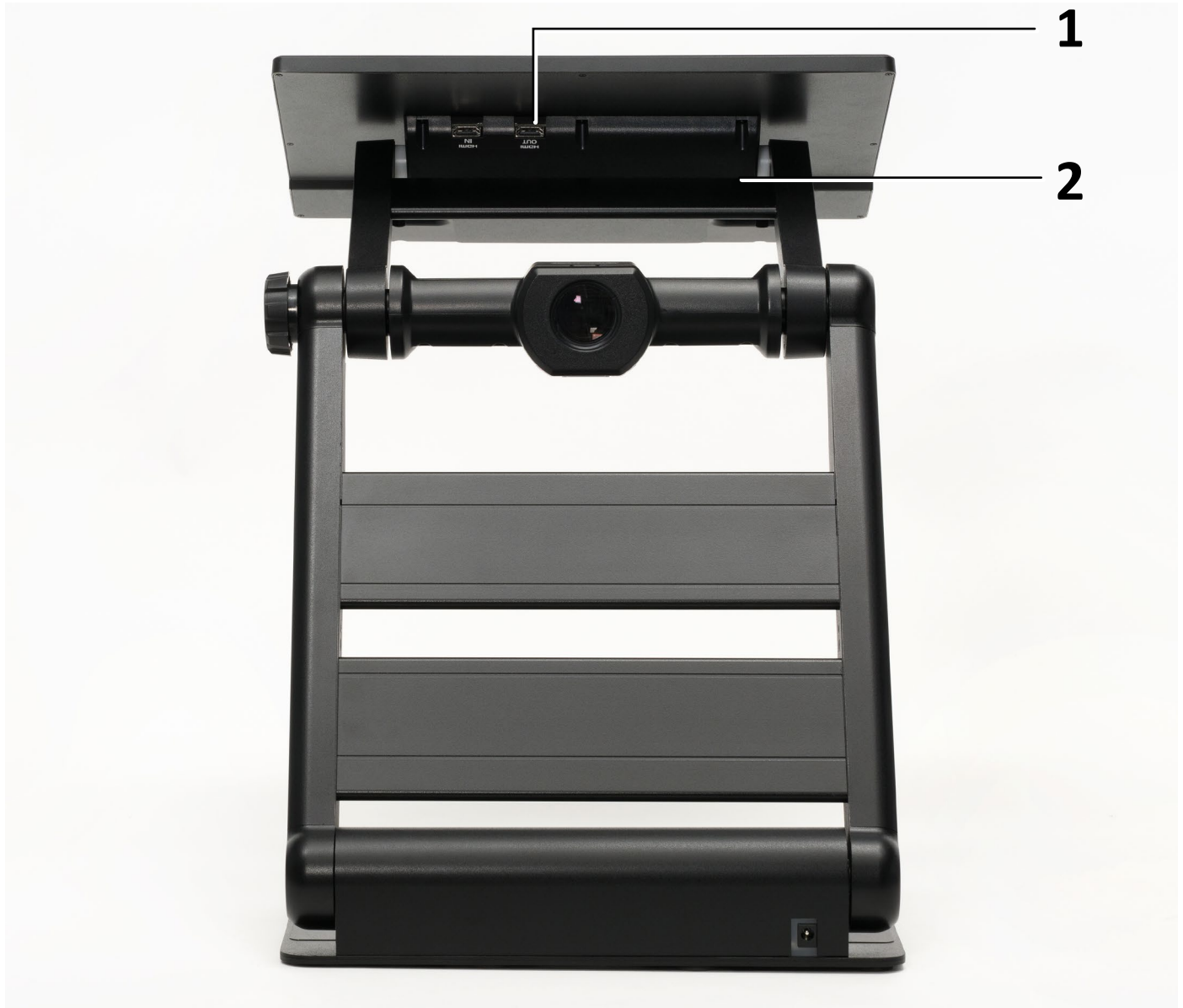

1. HDMI OUT
2. Test PORT (USB C type connector)

G. Configuration, Cont'd.



1

2

3

4

5

6

7

1. Power on/off button

- Power on/off: Press the button for 2 seconds.

2. Image optimizer wheel

- Provides two different functions, depending on the setting:
 - Natural color mode: Turn the knob to adjust brightness level of the display. (5 steps)
 - Color mode: Turn the knob to adjust the image quality optimizing settings. Press it to return to the default setting.
- Auto Focus on/off: Press and hold the button for 2 seconds.

G. Configuration, Cont'd.



3. Snap shot button

Briefly press the button and the image on the screen will freeze.

- You can then zoom in/out on the still image and adjust color and screen brightness.
- Press the button again to return to live mode.
- Super Function Key (7) scrolls the frozen image.

4. Zoom wheel

To adjust the magnification, turn the knob.

- Magnification ratio :
 - Live image mode: 1.8 x ~ 30 x
 - Snap shot mode: ~ 150 x
 - Adjustable magnification: 1.2 x ~ 250 x

Press and hold the button to use the menu mode.

G. Configuration, Cont'd.



1

2

3

4

5


6

7

5. Color mode button

The image on the screen can be viewed in eight (8) color modes

- Natural color mode
- White text on black background
- Black text on white background
- Yellow text on black background
- Black text on yellow background
- Green text on black background
- Blue text on yellow background
- Yellow text on blue background

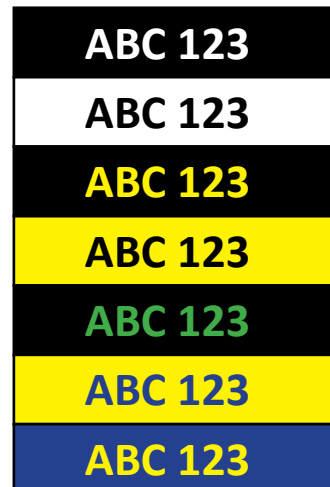

Up to 35 color modes can be set using the menu.

6. Natural color button

- Press the button to view pictures or documents in natural color.
- Press and hold the button to use the 'Find function'.

7. Super Function Key

Used for various functions in different modes.

- Scrolling the frozen image.
- Selecting/Using the line/mask functions

G. Configuration, Cont'd.



1. LCD screen


2. Camera

- Be careful not to allow foreign material to adhere to camera lens.

3. Camera location adjuster wheel

Adjust the direction of the camera by using the wheel.

G. Configuration, Cont'd.

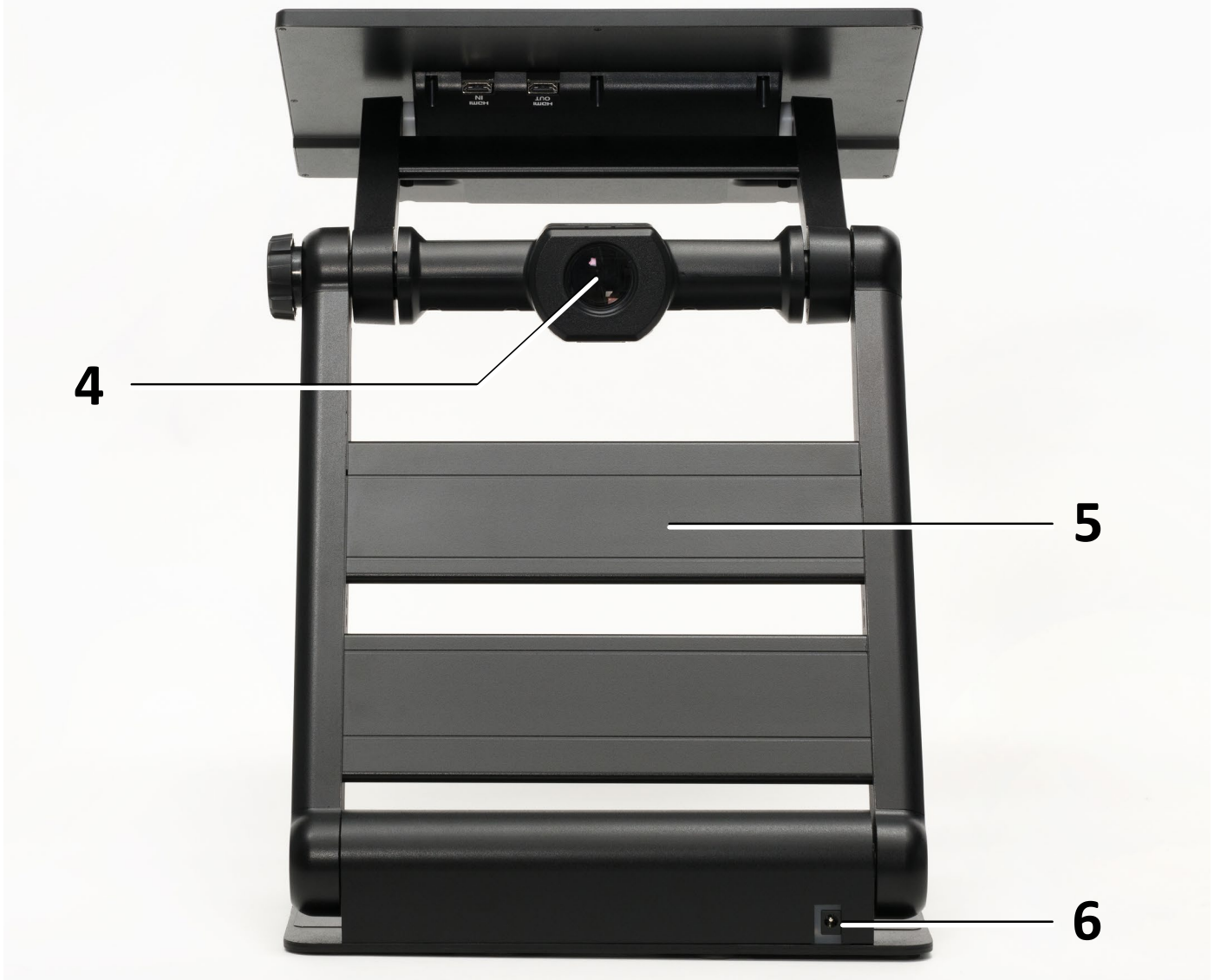

4. Conversion lens

Open the conversion lens for the distance view mode.

5. LED lights

6. DC jack

You can insert the power adapter here to charge the battery.

H. Hot Keys & Features


1. Line & Mask

Press and hold the **Super Function Key (7)** to activate Line & Mask.

When Line & Mask is active, the direction key and wheel of the **Super Function Key (7)** works as follows.

- Left & Right ◀▶: Choose the function.

1) Horizontal line

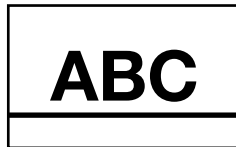


2) Vertical line



3) Horizontal mask



4) Vertical mask

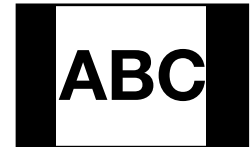


Up & Down ▲▼: Increase or decrease the thickness of a line and the distance between masks.




H. Hot Keys & Features, Cont'd.

- Wheel  : Move the position of the Line & Masks.

1) Counterclockwise

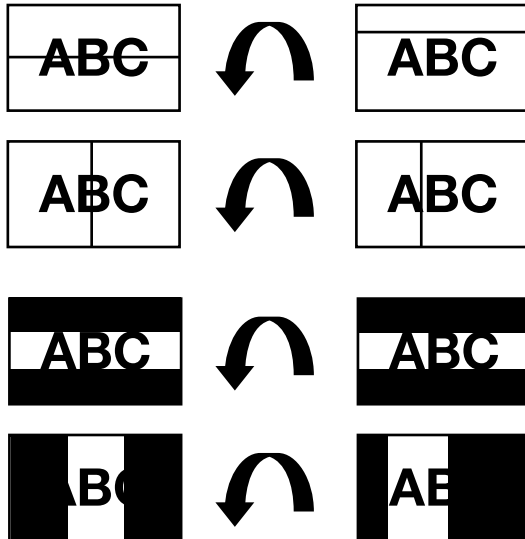

2) Clockwise

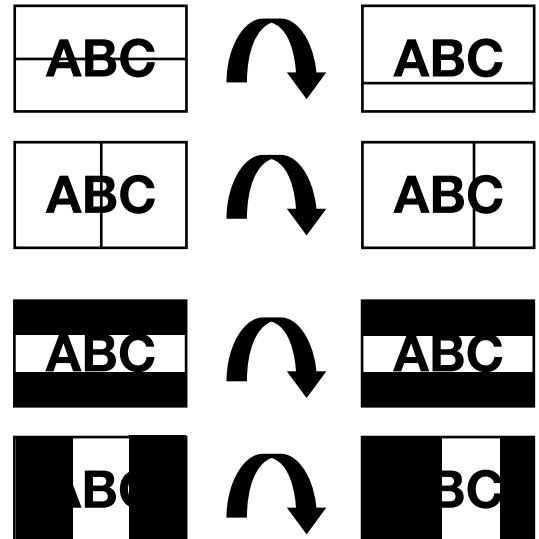

2. Battery status

- The status of the remaining battery power is not normally displayed.
- The battery charging icon when the power adapter is connected will be displayed.
- You can turn on or off the status of the remaining battery power by pressing the **Snap shot button (3)** for 3 seconds or more.

H. Hot Keys & Features, Cont'd.

3. Power saving mode

If you leave the unit turned on and unused for 10 minutes, it automatically goes into the Power saving mode. If the unit is left for 3 minutes after entering the Power saving mode, power is automatically switched off.

4. Color mode selection

Press and hold the **color mode button (5)** and rotate the **Zoom wheel (4)** to cycle through the available color modes.

- **Color mode button (5) + Zoom wheel (4, Right)**: Next color mode

Color mode button (5) + Zoom wheel (4, Left): Previous color mode

H. Hot Keys & Features, Cont'd.

5. Self-view mode & Distance-view mode

You can change the mode by moving the camera with the **Camera location adjuster wheel (13)**.

- Self-view mode (Mirror mode)

Use when the direction of the camera lens is facing the user.

- The camera switches automatically into self-view mode.
- The magnification range will be minimized.
- The camera switches automatically to natural color mode.

- Distance-view mode

Use when viewing a distant object, such as a whiteboard.

- The camera switches automatically into distance-view mode.
- The magnification ratio will be minimized.
- The camera switches automatically to natural color mode.
- Open the conversion lens when using the distance-view mode.

H. Hot Keys & Features, Cont'd.

6. Natural color & Find Function

- Press the **Natural color button (6)** to view pictures or documents in natural color.
- Press and hold the **Natural color button (6)** to zoom out and display the Find box on the screen to help you locate the object under the camera. Take your finger off the button to zoom in to the previous magnification level.

7. MENU

Press and hold the center of the **Zoom wheel (4)** to enter the menu mode. In Menu mode, you can change the settings of the following functions.

- Color mode : Add or remove your preferred color modes (max 35 modes)
 - You can also personalize your own color setting.
- Power saving mode
 - Adjust the time before entering power saving mode.
 - Default setting time: 10 minutes
- Adjusting the magnification range
 - You can set the magnification range.
 - Default magnification range: 1.8 x ~ 30 x
- Adjusting LED brightness
 - You can adjust the LED brightness.
- Reset
 - Resets all settings back to default values.

I. Plug&Play Smart Premium

1. Requirements

- Jupiter model that supports Plug&Play Smart Premium
- USB-C to HDMI Adapter Cable provided

The Plug&Play Smart Premium feature of Jupiter allows the user to connect portable devices such as phones and tablets to the Jupiter using the HDMI IN port. This feature allows the user to make use of the larger 13" LCD screen size, magnification features, and quality speakers.

2. Compatibility

- iPhone 15 or later
- iPad models with USB-C connectors
- Galaxy S20/S21/S22 or later
- Galaxy Note 8 or later
- Galaxy Tab S4 or later

3. Connecting

Insert the USB-C plug into the port of the portable device. Insert the larger HDMI plug into the HDMI IN port on the Jupiter located behind the display. The Jupiter will automatically detect and switch to the HDMI input within approximately 10 seconds of connecting. Disconnect the HDMI plug from the Jupiter to return to the camera view.

Note that the Jupiter will not charge your portable device when connected.

I. Plug&Play Smart Premium, Cont'd.

4. Displaying

If properly connected, the Jupiter will display the screen contents transmitted from the portable device. To make the most benefit of the wide screen of the Jupiter, rotate your portable device to the Landscape orientation.

Note that not all applications support Landscape layout. Some video streaming applications will stream directly to the Jupiter rather than displaying on your mobile device.

5. Zooming and Panning

The Jupiter can be used to magnify the images transmitted from the portable device. Turn the Zoom wheel knob right or clockwise to increase the size of contents displayed on the screen. Turn the Zoom wheel left or counterclockwise to decrease the size or to view the entire screen contents.

Note that when zoomed in, content will now be available outside of the viewable items on the screen. Use the Super Function key to pan the viewable area up, down left or right to view items that are outside the field of view.

6. Color Mode

The Jupiter can apply high contrast color modes to the images transmitted from the portable device. Use the color mode button to toggle through the preset color modes.

J. Technical Specifications

LCD Screen Size	13 inches (338mm), 16:9 wide screen
LCD Resolution	1920 x 1080 (Full HD)
LCD Rotation angle	180 degree
Camera Specifications	1920 x 1080, 60 fps Auto Focus

Dimensions	Approx. 13 inches (320mm) x 15 inches (373mm) x 17 inches (420mm) (in operation) Approx. 13 inches (320mm) x 15 inches (373mm) x 2 inches (61mm) (folded)
Light Weight	3.54 kg / 7.8 lbs. (without battery) 3.91 kg / 8.62 lbs. (with battery)
Color mode	Default 8 (Extended 35)
Magnification range	1.8 x ~ 150 x (Max)
Camera mode	Reading mode, Distance-view mode , Self-view mode

Battery capacity	16.8V, 5200mAh
Battery weight	370 g (0.82 lbs)
Power consumption	14~20W (in operation)
Battery lifetime	5 hours (Max)

Connectivity	HDMI OUT USB C type connector (Test PORT)
---------------------	--

J. Technical Specifications, Cont'd.

FCC Compliance Notice

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in some instances. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, you are encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment to a wall outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Unauthorized changes or modifications to this system can void your authority to operate this equipment.

Industry Canada (IC) Notice

This Class B digital apparatus complies with Canadian ICES-003.

Cet appareil numérique de la classe B est conforme à la norme NMB-003 du Canada.

This unit is an EMC-registered device.



This device bears the CE and FCC, KC approval mark.



American Printing House for the Blind
Louisville, Kentucky

Catalog No. 1-00370-01